For veterinary use only!

For quality control procedures of the quantitative determination with the Fructosamin (gold standard) VET test kit on a solo or CUBE-VET analyser

Eurolyser Diagnostica GmbH
Bindergasse 3
5020 Salzburg/Austria

Tel. +43 (0)662 / 43 21 00
Fax +43 (0)662 / 43 21 00-50
www.eurolyser.com

English

	Order information	Indication	Kit size
	Order number: VT 0110 Order number: VT 0111 Order number: VT 1100	Fructosamin (gold standard) VET test kit Fructosamin (gold standard) VET test kit Fructosamin VET control kit	16 tests/pack 6 tests/pack 1 x 1 ml (decision level)

Test kit preparation: Allow single test at least 10 minutes to warm up to room temperature (20 - 25 °C) by placing the test into the test kit rack. Put test kit package back into refrigerator.

Summary

This Fructosamin VET control kit is intended for use as a test sample of known concentration for the Fructosamin (gold standard) VET test kit. The user can compare observations with expected ranges as a means of assuring consistent performance of the Fructosamin (gold standard) VET assay.

Preparation and Handling

Control material is lyophilized and must be reconstituted before use!

1. Open the vial very carefully, avoiding any loss of the lyophilized material.
2. Add exactly 1ml of distilled/deionized water.
Warning: Inaccurate reconstitution of the control and error in assay technique can cause erroneous results.
3. Close the vial carefully and dissolve the components completely by occasional gently swirling. Avoid the formation of foam.
4. Allow the contents to reconstitute over a period of approx. 30 minutes.
5. Analyse the control in the same way as a patient sample.

Stability and Storage

Unopened vials are stable at 2 - 8 °C until the expiration date stated on the label.

Stability after reconstitution:

Stored at 2 - 8 °C: 2 weeks

Close immediately after use. Upon storage, caps should be screwed tightly.

Waste Management

Please refer to local legal requirements

Warnings and Precautions

This control kit is for veterinary diagnostic use only!
DO NOT INGEST! Exercise the normal precautions required for handling all laboratory reagents.
All human material should be considered potentially infectious. All products derived from human blood are prepared exclusively from the blood of donors tested individually and shown to be free from HBsAg and antibodies to HCV and HIV. The testing methods applied were FDA-approved or cleared in compliance with the European Directive 98/79/EX, Annex II, List A.
However, as no testing method can rule out the potential risk of infection with absolute certainty, the material should be treated just as carefully as a patient specimen. In the event of exposure the directives of the responsible health authorities should be followed.

Set the species to „Control“ and the sample type to „Ctrl.Plasma“ before running the QC material

Compare the results with the values indicated in the lot-specific value table

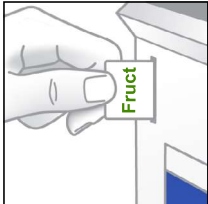
If the result is not within the acceptable range, repeat measurement. If result is again not within the acceptable range, please contact your distributor

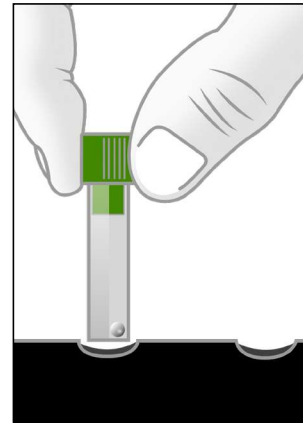
Durchführung einer Fructosamin VET Kontrollmessung Processing of a Fructosamin VET control measurement

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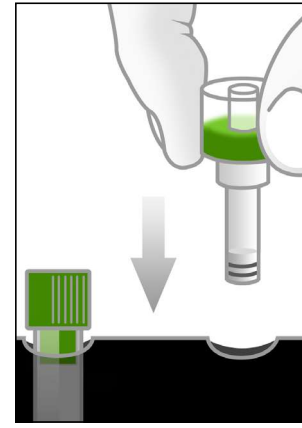
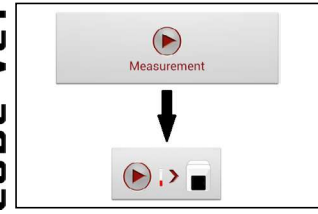
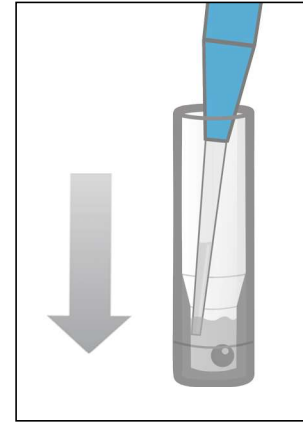
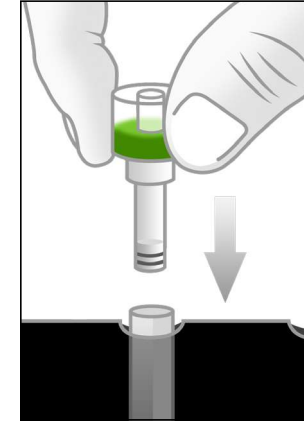
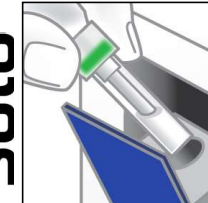
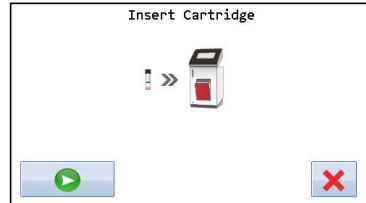
English

ATTENTION!

Allow single test at least 10 minutes to warm up to room temperature (20 - 25 °C) before use!

- 1. Preparation of test system**
 - 1.1 Place RFID card
 - 1.2 Place ERS cuvette in test kit rack
 - 1.3 Place ERS cap in test kit rack
 - 1.4 Press „Measurement“ button, enter required information using the touchscreen

- 2. Control preparation**

For further details please see user manual of laboratory photometer

 - 2.1 Set species to „Control“
 - 2.2 Set sample type to „Control/Ctrl. Plasma“
 - 2.3 Aspirate 20 µl control liquid out of control vial using a pipette

- 3. Control processing**
 - 3.1 Dispense 20 µl control liquid INTO THE LIQUID in the ERS cuvette
 - 3.2 Apply ERS cap firmly onto ERS cuvette
 - 3.3 Place ERS cartridge into laboratory photometer
 - 3.4 Start automatic control processing by pressing the  start button on the solo laboratory photometer, or by closing the door of the CUBE-VET laboratory photometer.

Wertetabelle für das Fructosamin VET Control Kit Value sheet for the Fructosamin VET control kit

LOT

2307-1

2024-05

Kontrollwerte | Control Values

Target	value (min)	value (max)
267.0 µmol/l	205.6 µmol/l	328.4 µmol/l

Technical details subject to change without notice.

Doc-ID: d1fea6df-90b8-4b77-b9ab-539a054b831c ; latest revision: DP 4.0 ; date of print: 2018-06-13

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