

## Vet ABC Plus Quality Control

### Save Target Values to USB Stick

**\*\* Very Important \*\* Make Sure Old QC files on the USB Key are deleted first**

1. Go to <http://www.vetnovations.com/service.html>
2. Left Click on Current Quality Control located under “Scil Vet abc Plus+ (English)” and select Save
3. Navigate to F: Removable to save file on the USB Key
4. Left click on the green arrow near the time and select “Safely Remove Hardware”
5. Remove USB Key from computer

### Load Target Values on ABC+

1. Put USB Key in Vet ABC Plus
2. Select the Quality Control menu
3. Select the appropriate column
4. Press the Upload Button
5. Select the Control you plan to run (Red, Green or Blue)
6. Press Upload again
7. Select the Keyboard, change the InterQC time limit from 24 to 9999
8. Press the X on the keyboard to exit
9. Press the big Green check mark at the bottom

### Running a Blood Control

1. Press the Sample ID Icon
2. Press the Keyboard and enter the Lot# you plan to run
3. If you entered the Lot# right, a purple Tube Icon should appear with QC on it.
4. Press the big Green check mark to begin the cycle
5. Place Tube under the needle and press the sample bar to aspirate the control.

### Deleting Old Control Target Values

1. Select the Quality Control menu
2. Select the column that hold the QC target values you want to remove
3. Press the Trash Can Icon and follow the prompts
4. Archive your changes when asked.

### Archiving QC Data

1. Press the Maintenance Icon; make sure your USB key is still in the machine.
2. Press the Arch. Tab at the top of the screen
3. Press the purple button beside QC
  4. Press the Partial Archive button (red and white disc) or the Reset Icon

